

SAM
MEDICAL

SAM[®] Pelvic Sling II

**SLIDE.
CLICK.
SECURE.**

OPTIMAL FORCE STABILIZATION

SAM[®] Pelvic Sling is the first and only force-controlled circumferential pelvic belt. Featuring an optimized one-piece design, it's scientifically proven to safely and effectively reduce and stabilize open-book pelvic ring fractures.



FEATURED TECHNOLOGY

AUTOSTOP BUCKLE

The patented SAM[®] AUTOSTOP buckle acts to avoid over- or under-tightening of SAM[®] Pelvic Sling. Once optimal compressive force is reached, two prongs activate, clicking to confirm correct application. The sling is secured in place with its hook and loop system strap.







SAM[®] Pelvic Sling II

ENGINEERED TO PRESERVE LIFE

- 1 LOW-FRICTION POSTERIOR SLIDER**
Easier transfers following severe trauma made possible by low-friction materials covering backside of the pelvic sling.
- 2 SAM[®] AUTOSTOP BUCKLE**
Tactile and audible “click” provides clear feedback, confirming proper application and preventing over-tightening while maintaining optimal force.
- 3 PRECISION STRAP**
Precisely-engineered holes lock with AUTOSTOP Buckle, allowing for quick application. Secure fastening with sturdy hook and loop strap.

- 4 TAPERED DESIGN**
Narrow and tapered front facilitates urinary catheterization, interventional radiology, external fixation, and abdominal surgery without removal.
- 5 RADIOLUCENT MATERIAL**
All materials are radiolucent except the buckle’s two stainless steel springs, allowing X-rays and CT-scans without removal.

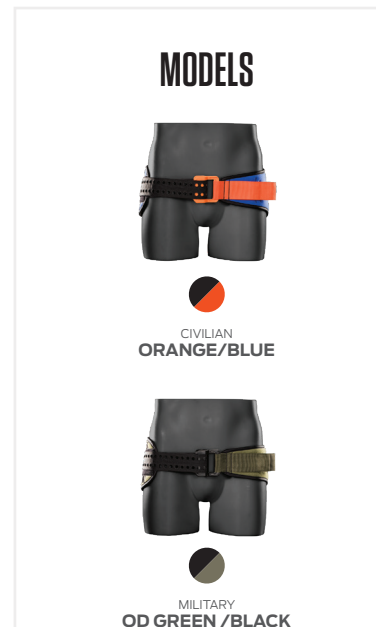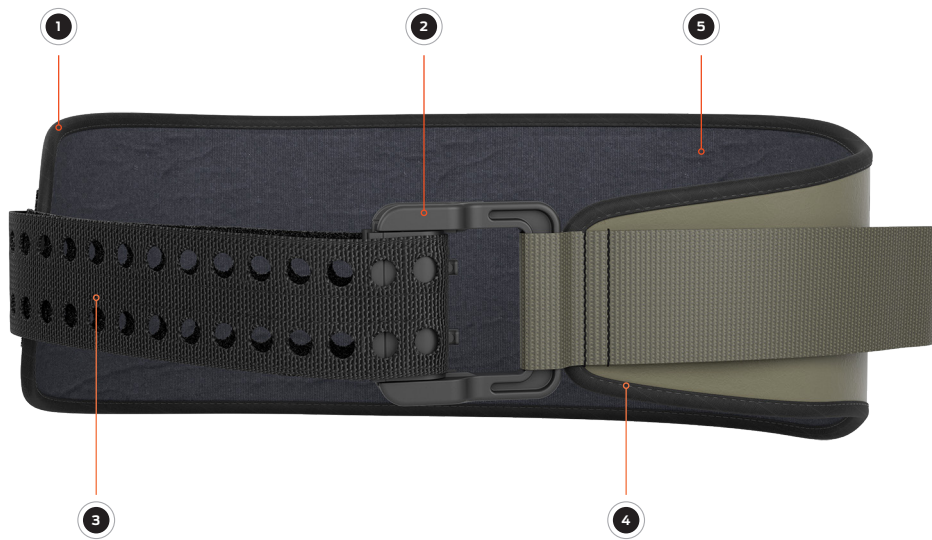

PRODUCT SPECIFICATIONS

PACKAGED SIZE & WEIGHT*

Height: 5.5 in (14 cm)
 Width: 7.5 in (19 cm)
 Depth: 3.75 in (9.5 cm)
 Weight: 8.6 - 9.2 oz (244 - 250g)**

OPERATING PRINCIPLE

Optimal force is reached when 150N of circumferential pressure is attained, which extends prongs from SAM[®] AUTOSTOP Buckle through the Precision Strap.

MODELS AVAILABLE

Military
 Standard 32-50" / 81-127 cm (PS301-OD-EN)
 Civilian
 Small 27-45" / 69-114 cm (PS300-OB-EN)
 Standard 32-50" / 81-127 cm (PS301-OB-EN)
 Large 36-54" / 91-137 cm (PS302-OB-EN)

GLOBAL SALES

U.S. Toll Free: 800.818.4726
 U.S. Local: 503.639.5474
 U.S. Fax: 503.639.5425
sammedical.com

Ergo Europe, Prinsessegracht 20, 2514 AP, The Hague, The Netherlands	SAM Medical Products [®] 12200 SW Tualatin Rd., Ste 200 Tualatin, OR 97062 USA	

* Size and weight vary by configuration and manufacturing process.
 ** Weight varies by size (Small, Standard, Large)
 SAM[®] is a registered trademark in the United States and other countries.
 ©2019 SAM Medical Products. All rights reserved.